

DECON Reeves Tent Setup

1. Lay out tarp to protect tent.
2. Lay out the berm (black tarp looking)
 - a. Sides – Collapsed and flipped under
 - b. Entrance/Exit – Collapsed and flipped forward
 - c. Yellow safety markings facing upward
3. Put tent in middle of berm
4. Remove 4 white tent poles
5. Wire loops in row indicate side walls
6. Loosen straps
7. Have one person at each corner
8. Lift and expand shelter by walking backwards
9. Flip doors and side fabric over so they are hanging down
10. 2 red tab lifting points are located on each end
11. Lifting together, place white poles under red tabs for support (they do not insert into holes)
12. Center shower wands, shower heads and lights can be installed:
 - a. The 2 red ones are inject solution lines
 - b. The 2 black ones are rinse lines
 - c. These are installed for non-ambulatory patients. Red is entrance, black is exit.
13. Now lift shelter with poles until side walls are level on ground. There is no click when extended up.
14. Berm walls can now be set up around side walls up around. Side walls up first, one entrance left down for setup.

Grids:

- Place down grids on both side of rollers and all flooring all around.
- Put up entrance berm
- Place submersible pump inside at lowest corner and attach discharge hose
- Install lane curtains and anchor cords to grid

DECON Tent Teardown

Let tent dry out before storing:

1. Remove Curtains
2. Remove wands and showerheads and replace with caps
3. Remove discharge hose from pump and remove pump from berm
4. Remove grids and rollers
5. Collapse side walls under
6. Collapse entrance/exit wall forward
7. Lift corners of shelter and flap over berm so it can be collapsed
8. Lift side walls over berm
9. Locate striking positions (2 each side). Place hand under flap
10. SAFETY: No one inside can be inside and ensure before collapse
11. Lift and step back to strike out for collapse
12. Lift shelter and walk inward
13. Place straps (2) around and cinch up
14. Place on side and re-cinch
15. Put bag over shelter after standing shelter up
16. Lay down shelter and put poles in bag
17. Cinch tie on top of box
18. Tighten outside straps
19. Fold berm length-wise in quarters.

Collapsible Rollers:

- Expand and center
- Adjust height, locate second locking, slide up ring, turn knob inside to tighten.
- Turn wheels inside.